**Below are KN95 User Instructions:**

**3-Steps Before Use:**

1. Do not use if the packaging has been opened or damaged
2. Check that ear loops are in good condition and are not damaged
3. Check that the metallic strip is not broken

**Wearing Instructions:**

1. Hold the ear loops in both hands with the metallic strip up
2. Put the respirator against your face, covering BOTH nose and mouth.
3. Place ear loops around both ears
4. Adjust respirator over nose and under chin
5. Adjust the metallic strip over bridge of nose
6. Make sure the respirator fits properly, covering the NOSE, MOUTH and CHIN.

**Fitting Test:**

1. Place both hands over the respirator
2. Inhale vigorously and hold your breath for a few seconds, making sure the respirator collapses inward as you inhale
3. Exhale vigorously and hold your breath for a few seconds, making sure the respirator bulges outward as you exhale
4. If air leaks, reposition the respirator, tighten the metallic strip and reposition the ear loops for a better fit, then follow steps 1 – 3 again until a tight seal is achieved
5. Respirators that have passed the fit test in steps 1-3 are safe to use
6. Facial hair or other facial features may prevent the respirator from achieving a proper fit. If a tight seal is not achieved, please do not use this product.
7. If you experience difficulty in breathing, dizziness, or other symptoms, leave the contaminated area immediately

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